



clinigo<sup>®</sup>

USER MANUAL

# Clinical Disinfectant Universal Wipes

Disposable Cleaning Wipes for Disinfecting Surfaces and Non-invasive Medical Devices



# Clinical Disinfectant Universal Wipes

50  
WIPES

100  
WIPES

200  
WIPES

## IDEAL FOR USE IN HOSPITALS AND HEALTHCARE SETTINGS.

A ready-to-use, single-use, 2-in-1 wipe for the one-step cleaning and disinfection of non-porous surfaces and non-invasive medical devices in a healthcare environment. The wipes are intended for use by healthcare professionals to reduce microbial contamination (including bacteria, fungi, and viruses), thereby supporting infection control and prevention protocols.

### THE ESSENTIALS

- Bactericidal
- Tuberculocidal
- Virucidal
- Fungicidal
- Yeastocidal
- Alcohol free
- Ready to use
- Gentle on surfaces

### COMPATIBLE MATERIALS

Compatibility with common healthcare equipment materials such as:

- Polycarbonate
- Acrylonitrile Butadine Styrene (ABS)
- Polymethyl Methacrylate (PMMA) (Acrylic)
- High Density Polyethylene (HDPE)
- Polypropylene
- Stainless Steel

### IDEAL FOR

- Effective cleaning and disinfection in a short contact time
- Intermediate disinfection where there is a known or a risk of infection\*
- All healthcare environments; available in flow wraps
- Disinfecting surfaces of non-invasive medical devices and equipment

**\*Not intended for end point sterilisation of medical devices.**

## INGREDIENTS

< 5% Non-ionic Surfactants,  
< 5% Cationic Surfactants,  
Disinfectant, Parfum.

### ACTIVE SUBSTANCE PER 100GRAMS OF PRODUCT:

Didecyldimethylammonium chloride (DDAC) 0.86g/100g,  
Alkyl (C12-16) dimethylbenzyl ammonium chloride (ADBAC/BKC (C12-16)) 0.8g/100g,  
N-(3-aminopropyl)-N-dodecylpropane-1, 3-diamine (Diamine) 0.7g/100g).

## CLINICALLY PROVEN

### CLINICALLY PROVEN TO KILL 99.99% OF GERMS.

- Including:
- Norovirus
  - Coronavirus
  - MRSA | VRE | CRE
  - + many more



### Saraco Industries Ltd

Egerton Street, Farnworth  
Bolton BL4 7ER  
United Kingdom  
T : +44 (0) 870 803 3979 UK

# Clinical Disinfectant Universal Wipes

50  
WIPES

100  
WIPES

200  
WIPES

## DIRECTIONS FOR USE AND CONTRA-INDICATIONS

Use gloves and appropriate PPE. Dispense wipe and open flat to clean the surface, wipe the surface in an 'S' shape motion from top to bottom. To ensure the surface remains wet for the required contact time, if necessary, use additional wipes. Allow the surface to air dry naturally before use.

## CONTRA-INDICATIONS

Do not use wipes for end point sterilisation of medical devices.


### STEP ONE

- Use gloves and appropriate PPE
- Follow your organisation's risk assessment policy when using PPE



### STEP TWO

- Open lid and label
- Pull out wipe
- Close lid re-sealing pack

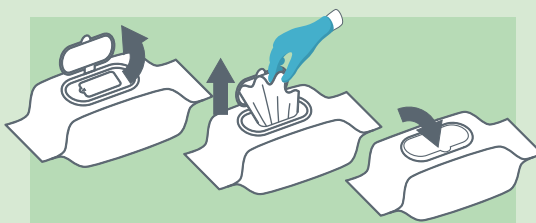

### STEP THREE

- Clean surface in 'S' shape motion, working from clean to dirty areas
- Keep the wipe flat (not scrunched) to ensure full surface coverage
- Avoid wiping the same area twice



### STEP FOUR

- Dispose of wipe in appropriate bin
- Do not re-use wipes
- Allow the surface to dry naturally before use
- Leave the surface for the stated contact time



## USE BIOCIDES SAFELY

Always read the label and product information before use.

Re-use of wipes increases the risk of infection, discard each wipe after single use.

## FOR PROFESSIONAL USE ONLY

### WARNINGS

- Harmful to aquatic life with long lasting effects.
- Keep out of reach of children.
- Avoid release to the environment.
- Dispose of contents/container to a licensed facility in accordance with national regulations.
- Always read the label before use. Use biocides safely and sustainably, this product should be used in accordance with the product label
- Wash hands thoroughly after use.
- If irritation occurs, stop use immediately and rinse with clean water, if symptoms persist seek medical advice
- Do not use with other cleaning products
- Not for skin cleansing or other personal use
- Do not flush down the toilet.

## TO REPORT A POISONING INCIDENT

If a serious incident occurs in relation to this device, the user must report it to the manufacturer and the competent authority of the Member State where they are established.

**National Poisons Information Centre**  
(01 8092166) (IRE)  
8am -10pm every day.

**For Safety Data Information**  
(medical personnel only)  
T: 08708033979 (UK).



**Saraco Industries Ltd**  
Egerton Street, Farnworth  
Bolton BL4 7ER  
United Kingdom  
T : +44 (0) 870 803 3979 UK

# Clinical Disinfectant Universal Wipes



## EFFICACY DETAILS

The liquid extracted from the wipes have been tested against: EN16615, EN14476, EN13727, EN13624, EN14348 and EN14562 under dirty conditions.

EFFECTIVE AGAINST	TEST*	CONTACT TIME SECONDS
<b>BACTERICIDAL</b>		
Escherichia coli	EN 13727	30
Carbapenem-Resistant	EN 13727	60
Enterobacteriaceae (CRE)	EN 16615	30
Vancomycin-Resistant	EN 13727	30
Enterococci (VRE)	EN 16615	30
Enterococcus hirae	EN 13727	30
	EN 16615	30
Klebsiella pneumoniae	EN 13727	30
	EN 16615	30
Pseudomonas aeruginosa	EN 13727	30
	EN 16615	30
Methicillin-resistant	EN 13727	60
S. aureus (MRSA)	EN 16615	30
Staphylococcus aureus	EN 13727	30
	EN 16615	30
<b>MYCOBACTERICIDAL</b>		
Mycobacterium avium	EN 14348	120
Mycobacterium terrae	EN 14348	60
<b>VIRUCIDAL</b>		
Murine norovirus	EN 14476	30
Vaccinia virus	EN 14476	15
Hepatitis B Virus (HBV)	EN 14476	15
Hepatitis C Virus (HCV)	EN 14476	15
Influenza Virus	EN 14476	15
Coronavirus (SARS, MERS)	EN 14476	15
Human Immunodeficiency	EN 14476	15
Virus (HIV)		
<b>YEASTICIDAL</b>		
Candida albicans	EN 13624	60
	EN 16615	30
	EN 14562	120
Candida auris	EN 16615	60

\*TESTED UNDER DIRTY CONDITIONS

## PACKAGING SYMBOLS

-  Do not re-use wipes.
-  Dispose of wipe in appropriate bin.
-  Use pack within 2 months of opening.
-  Store in a cool, dry place out of direct sunlight.
-  Temperature limits for transport and handling.
-  REF CU8690
-  MD UFI: XT70-D0GM-R00D-87T7
-  Available from: [www.avrahealthcare.com](http://www.avrahealthcare.com)

## FOR INFORMATION

For Safety Data Information (medical personnel only)  
T: 08708033979 (UK).



**Saraco Industries Ltd**  
Egerton Street, Farnworth  
Bolton BL4 7ER  
United Kingdom  
T : +44 (0) 870 803 3979 UK

# Clinical Disinfectant Universal Wipes

50  
WIPES

100  
WIPES

200  
WIPES

## PRODUCTS WITHIN THIS RANGE

### 50 Clinical Disinfectant Universal Wipes

Wipe Size:	200 x 200 mm
Substrate:	30gsm
Packs Per Case:	24
Product Code:	CU8676
Barcode:	5060110228676



### 100 Clinical Disinfectant Universal Wipes

Wipe Size:	200 x 200 mm
Substrate:	30gsm
Packs Per Case:	12
Product Code:	CU8683
Barcode:	5060110228683



### 200 Clinical Disinfectant Universal Wipes

Wipe Size:	200 x 270 mm
Substrate:	30gsm
Packs Per Case:	6
Product Code:	CU8690
Barcode:	5060110228690



**Saraco Industries Ltd**  
 Egerton Street, Farnworth  
 Bolton BL4 7ER  
 United Kingdom  
 T : +44 (0) 870 803 3979 UK